

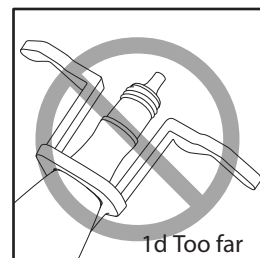
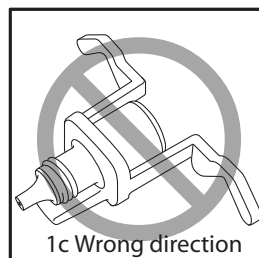
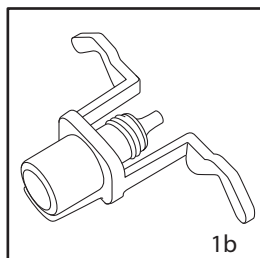
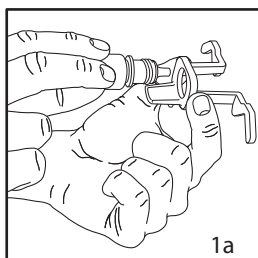


The DROPin™




(Get the drop in)

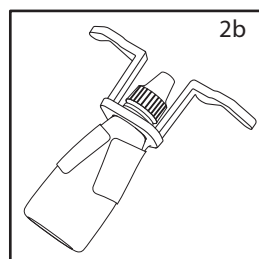

1. Remove the cap and place the bottle* through the hole of the DROPin™. The bottle tip is pointed toward the triangular ends and not going past them.

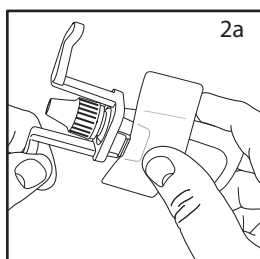

2. If the bottle does not snap or fit snugly into the hole, wrap one of the enclosed stickers around the bottle and the DROPin™. If you run out of the enclosed stickers, any tape should suffice or you can email us to order more stickers at info@ocunelis.com.

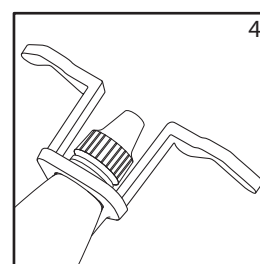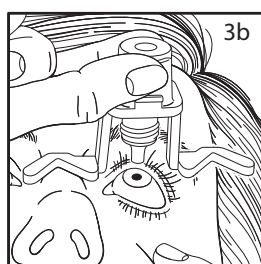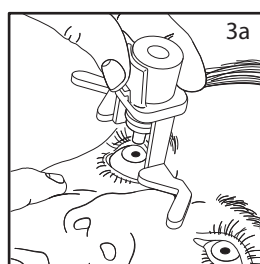

3. Using one hand, bring the DROPin™ to your face with the triangular end cupped around the top of your nose — centering the drop tip over your eye. Lean your head back. With your free hand, you can hold open your eye or lower the eyelid. Squeeze the bottle with one hand to apply a drop. If needed, shift the DROPin™ to the other eye and repeat.
4. Replace the cap onto the bottle with the DROPin™ still attached. Some bottle types need to be pulled out of the DROPin™ in order to re-cap the bottle.

Please provide us feedback at www.ocunelis.com

*Bottle not provided



DISCLAIMER: Use device only as directed. No contact with your eye, eye lid, or eyelashes should be made with the DROPin™ or the eye drop bottle.

